

# **HEARING AMPLIFIER 02001 USER MANUAL**



**Please read this manual completely  
before using this product.**

## ***Functions***

Thank you for purchasing this product.


When watching quiet films or plays, attending meetings, or sitting in a classroom, this product can help you hear things that you might otherwise miss.

For outdoorsmen and birdwatchers, this product can enable you to enhance your sense of hearing and pick up clues in the environment.

## ***Specifications***

1. Max. sound output: 130 +/- 5dB
2. Sound gain:  $\geq 50$ dB
3. Total harmonic wave distortion:  $\leq 5\%$
4. Frequency range: 300 - 4000 Hz
5. Input noise:  $\leq 30$  db
6. Voltage: 1.5V
7. Current:  $\leq 4$  mA

## Diagram

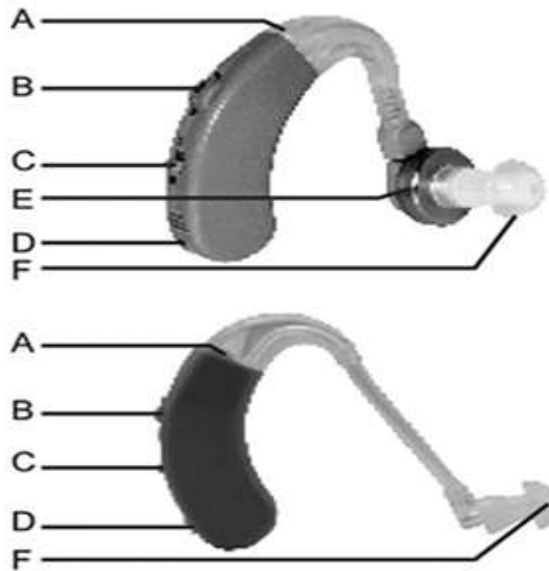

- A. Sound hole
- B. Volume control
- C. On / Off switch
- D. Battery compartment
- E. Earphone
- F. Ear piece

"N" means "ON"

"O" means "OFF"

"+" is the positive electrode of the battery

"-" is the negative electrode of the battery

"1 - 4" is the volume index. "1" is minimum,

"4" is maximum.

## **Usage**

Slide the battery compartment open and insert the battery in the correct orientation.



Use an LR754 size alkaline battery.

1. Before use, make sure power is OFF and volume is set to "1".



2. Place an earpiece on the earphone (3 different size earpieces are included - select the one that fits snugly). Position the body of the device behind your ear and place the earpiece into your ear, being careful not to push it in too deep.

3. Turn on the power and adjust the volume as needed.



4. If you hear a whistling sound, check that the earpiece is snug. If not, change to a different size earpiece.



## ***Troubleshooting***

No sound

1. Make sure the power is on.
2. Make sure the battery is inserted in the correct orientation.
3. Make sure the battery is not drained.
4. Make sure the device is not damaged.
5. Make sure the lead between the device and the earpiece is not broken

Weak sound

1. Make sure the battery is not drained.
2. Make sure the earpiece is well seated and not blocked.

## ***Important notices***

1. This is not a medical device and is not intended to cure hearing loss.
2. Do not use if you have hearing problems or ear pain.
3. Do not use at high volume or for an extended time. If you feel any discomfort, discontinue use immediately.
4. Keep the earpiece clean and dry.
5. Not intended for children. Small parts are a choking hazard.

**Clean with a soft, dry cloth.**

## ***Battery***

This device uses one LR754 size alkaline button cell battery (1.5V). (Two batteries are included.)